


4. Switch on start/pause button [10] and the working indicator [7] lights. The corresponding intensity indicator lights.
5. Press scrubber output intensity selecting button [11] to select certain intensity.
6. If need pausing during operation, please press pause button [10].
7. After operation, switch off the power and unplug the power line. Clean the accessories.

### III. Usage


The beauty method of skin scrubber is the way of contacting the skin directly that drives the probe as drawing the straight line on the face. The output mode is SERISE (continuous wave "C")/PULSE (impulse wave "P"). "SERISE" is the continuous wave that meant the energy of sound wave is sending without resting. The function of this kind of sound wave is equality, thermal effect is obvious that suitable for the normal skin. "PULSE" is the impulse wave that has the rhythm energy of the sound wave launching. The time of the sound wave is much shorter, and the beam is launching regularly. The specification is that use the strong penetrating force and beam to exert the mechanical action based on the reducing the thermal effect. It is suitable for someone has the thick stratum, and the skin is roughness.

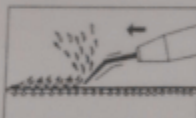

A. Remove the aged cutin

#### A. Operating Principle

The energy converting technology proper to the skin scrubbing instrument provides mechanical vibration at 25,000 Hz, emitting energy to exert remarkable osmosis on human skin. Through atomized conduction by means of medium, it can produce strong impact to remove the aged cutin and clean deeply, making the skin smooth and tender.

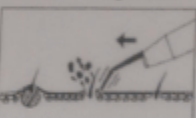

B. Remove the acne and comedo

#### B. Operating Principle


The ultrasonic vibration for 25,000 Hz per second makes the purified water pulverization that can clean the skin deeply, remove fouling in the pores instantly and expel acne and whelks in the pores indolently to generally clean the skin.



C. Remove the wrinkle

#### C. Operating Principle

The vibration of ultrasonic and warm heat effect from ultrasonic can make the skin fibre more active, and nutrients and water deeply conduct in by supersonic to improve the slack skin and remove the wrinkle.

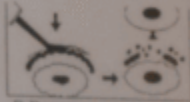

D. Decompose the melanin

The vibration of energy of the ultrasonic can go deep into the bottom of the epidermis, remove fouling in the pores instantly and make melanin less solvable.

### IV. Attention

1. While the skin scrubber is moving, the action must be fast. Do not operate on the same part for so long time. Must make sure the medium is enough.
2. Do not close to the eye and the throat.
3. Do not turn on the skin scrubber for a long time without anything to operate. When the skin scrubber leaves away the client's face, must stop the machine.
4. Please pay attention to the components and instrument that have to be keep away from the water.
5. Have to rest for 15-20 minutes after continuous using for 15 minutes, when you keep using it.

### Ultrasonic

#### I. Functions

1. Improve veins and remove the wrinkle.
2. Accelerate the lymph's circulation, and regain the skin radiance and elasticity.
3. Recover the fibrous tissue's elasticity.
4. Make the skin absorb the nutrition to improve the skin radiance.
5. Lighten the melanin and color spot.
6. Remove the black eye socket, pouch.

#### II. Operation methods

1. Plug the ultrasonic into the corresponding socket [45].
2. Clean face skin with facial scrub lotion and steam face with steamer vapor for 10 min before starting this function.
3. Press the ultrasonic mode selecting button [17] to choose certain mode. Their corresponding mode indicator will light.
4. Press Ultrasonic flat/sharp probe switch button [18] to select FACE (flat probe) or EYE (sharp probe). Their corresponding indicator will light.
5. Press start/pause button [23] and the working indicator [24] lights. The function is at working condition.
6. Press intensity adjusting button [20] to set the intensity strength.
7. If need to pause during operation, please press pause button [23].
8. After operation, switch off the start/pause button [23] and cut off power supply and clean probes.

### III Usage

KIN

### BEAUTY MACHINE

1. Lead the nutrition deep into the skin, whiten the skin, and lighten the color spot:

Insert the flat probe into the relative plug. And adopt the way of circling moving to cling the skin from down to up, from the corner of the mouth to front eyebrow, from nosewing to temple, from the top of the eyebrow to hairline with the medium intensity. 20 mins per once, 1-2 times every week, ten times to be a period of treatment.

This method is suitable for enhancing the permeability of the hypodermic cell, make the skin smoother, decompose the melanin and lighten the color spot.



2. Modify the outline, remove the wrinkle

Use the flat probe with the medium intensity to cling the skin in the style of instituting and pulling to move as the straight line from chin to earlap, from the corner of the mouth to the top of ear, from nosewing to the temple and from top of the eyebrow to the hairline, and use the other hand to coordination the flat probe to make the straight moving. 20 mins per once, 1-2 times every week, ten times to be a period of treatment.

This method can regain the radiance skin and improve the lymph's circulation and lead the loosen skin, remove the wrinkle.



3. Eye treatment

Use the sharp probe to move as drawing circle or line in slowing and lighting way from outside to inside, from the inside corner of the eye to the outside corner of it with the low intensity. 10 minutes per one time, 2 times every week, ten times to be a period of treatment.

This way is good for the blood circulation of the eyes; eliminate the pouch and black eye socket.



Beautifully and set up the suitable mode and treatment according to the client's skin.

### BEAUTY MACHINE

KIN

Adaptation Part	Mode	Probe	Power	Time
Modify the outline, improve the skin, and remove the wrinkle	100% 75% 50%	Flat Probe	4-8 lights	20 min/ time
Lead the nutrition deep into the skin, whiten the skin, and lighten the color spot and melanin	100% 75% 50%	Flat Probe	4-8 lights	
Eye treatment	100%	Sharp Probe	3-6 lights	10min/ time
Sensitive and uneven skin	75% 50%	Flat Probe	3-6 lights	8-10min/ time

### IV Attention

1. Cleanse the skin carefully and rub the enough medium, and then the probe can be started.
2. The heating level of probe is relating to the energy output, but, only if warm heat that is enough.
3. Have to choose the suitable intensity output if the client's skin belongs to the sensitive skin. When the client feels uncomfortable, must adjust to low down the intensity.
4. The probe can not operate without target for a long time, otherwise, it can be damaged because of superheating. If need to stop it, please press the key <stop>.
5. Put a little bit water on the probe while it is working, and then adjust the output intensity gradually. Observe the working condition and power of instance of quaking of the probe through the water granules.
6. The probe can not operate on these place directly: human's heart, throat and upper eyelid.
7. Purely- natural water-soluble liquid is selected. The rate of probe moving is 0.5-2cm per second. The probe can not be used without any medium.

### Hot/cold hammer

#### I. Functions

1. Improve the lymph's circulation and prevent senescence.
2. Eliminate the Blood Stagnation and dropsy.
3. Supply the moisture and nutrition of the skin's demand.
4. Remove the black eye socket and pouch.
5. Ease the reaction of the sensitive skin.
6. Lymph drainage and remove the toxin, and whiten the skin.

#### II. Operation methods

1. Plug the hot/cold hammer into the output socket [47].
2. Clean face skin with facial scrub lotion and steam face with steamer vapor for 10 min before starting this function.
3. Press the hot/cold hammer start/pause button [37] to start the function and its